

Quick Reference Guide

Daily Patient Assessment

DIGNISHIELD™

STOOL MANAGEMENT SYSTEM

Cuff Placement:

- Assess position of the green transphincteric zone.
- Ensure the cuff has not moved from previous positioning.

Cuff Volume:


- Ensure cuff has been inflated with 45 ml of water.
- Cuff volume should not exceed 45 ml of water.

Patient Care:

- The device is not intended for use beyond 29 days.
- Do not insert anything into the anal canal while this device is in place (e.g. thermometer, suppositories, etc.). Remove the device prior to insertion of anything into the anal canal.
- The device is not indicated for medication delivery.
- Do not use on patients with any rectal or anal injury, severe rectal or anal stricture or stenosis (or any patient if the distal rectum cannot accommodate the inflated cuff), confirmed rectal or anal tumor, severe hemorrhoids or fecal impaction.
- Do not use on patients known to be sensitive to or allergic to any components within the system.
- Perianal care should be performed at routine intervals to ensure the area is clean and dry.
- If stool consistency becomes more solid, device use should be discontinued.
- Consult Instructions for Use for further information on the use of this device.

Device Maintenance:

- Flush the catheter using the purple FLUSH port as needed to remove residual fecal matter from the tube. Note the amount of fluid infused.
- The patient's bowel can be irrigated using the clear IRRIG port.
- When changing a full collection bag, secure the bag plug and dispose in accordance with hospital protocols.
- Check position of the green transphincteric zone after repositioning patient to ensure the device is seated properly within the rectal vault.
- Check the collection bag at least once per shift to monitor stool volume.
 - If stool volume decreases significantly, check the catheter for potential kinks / blockages
 - If stool consistency becomes more solid, device use should be discontinued.

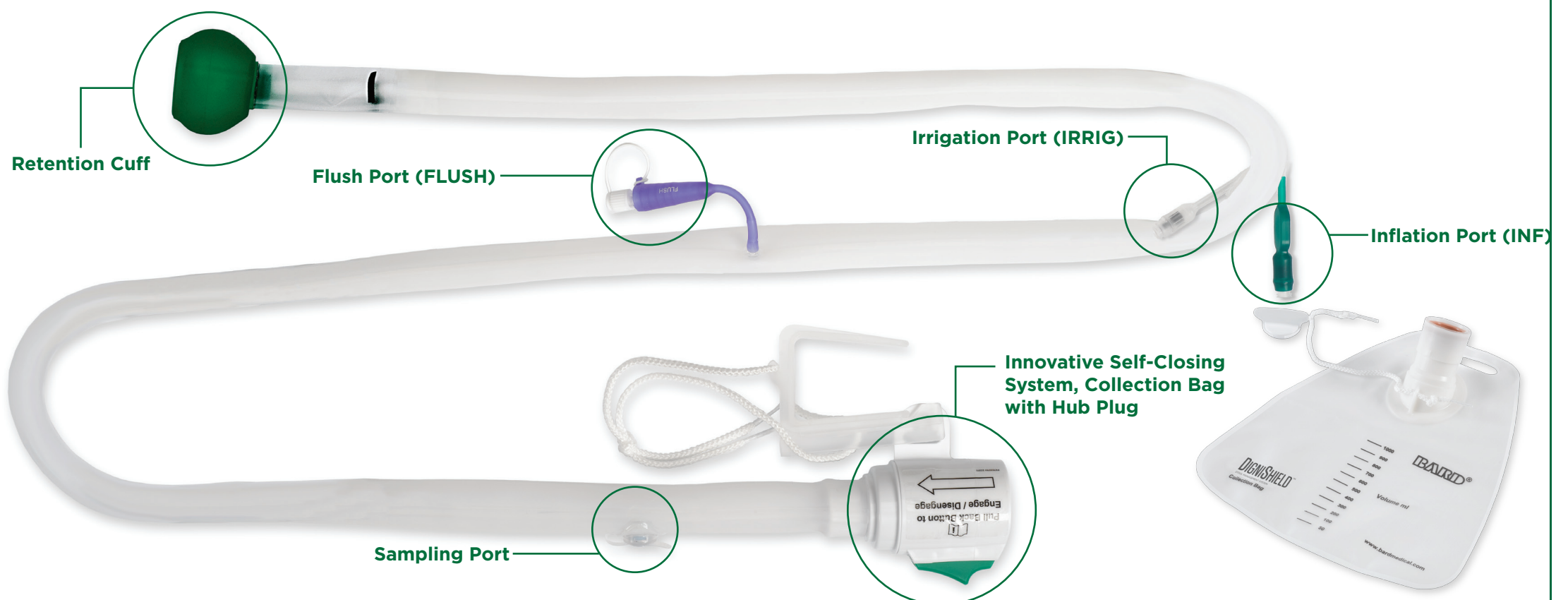

Leakage:

- Assess leakage around the perianal area.
- Ensure the catheter is unobstructed and that there are no kinks or twists in the tubing.

Flush System:

- Using the purple FLUSH port, irrigate the catheter regularly to keep the tubing clear and minimize odor.

Leakage:

- Ensure catheter is not kinked, twisted or externally compressed. If this has not reduced leakage, go to the next step:
 - Using a syringe, withdraw water from the green INF port to assess cuff fluid volume.
 - Reposition the patient and continue to withdraw water from cuff. This ensures all water has been removed.
 - Fill the syringe with 45 ml of water and re-inflate the cuff using the green INF port. **CUFF VOLUME SHOULD NOT EXCEED 45 ML OF WATER.** This will cause the cuff to become occluded.
 - Gently pull on the catheter tube to ensure cuff seating in the rectal vault
 - Irrigate the bowel using the clear IRRIG port to determine if above steps have assisted in minimizing leakage
 - If leakage persists, deflate the retention cuff and remove the device. Wipe the unit with a disposable towel and re-insert the device following the instructions for use.

Odor Control:

- Flush the catheter using the purple FLUSH port at least twice per shift and as needed. Milk the tubing to as needed to drain any residual fecal matter into the collection bag.
- Replace collection bag routinely. Secure the bag plug and remove from room. Dispose in accordance with hospital protocols.
- Check position of the green transphincteric zone after repositioning the patient and at regular intervals to ensure the device is seated properly within the rectal vault.
- Irrigate the bowel with water using the clear IRRIG port to ensure proper drainage.

Bard and DigniShield are trademarks and/or registered trademarks of C. R. Bard, Inc. or an affiliate. Please consult product label and insert for any indications, contraindications, hazards, warnings, cautions and directions for use.

©2011 C. R. Bard, Inc. All Rights Reserved. 1111-31 R11/11 THP P12/11 5M

C. R. Bard, Inc.
Covington, GA 30014 USA
1.800.526.4455
www.bardmedical.com

BARDD | MEDICAL