

Label Legend

If your patient needs a Foley catheter, reach for the tray that matches your patient assessment.




DRAINAGE BAG

A regular pre-connected closed system Foley catheter tray.



URINE METER TRAY

If the close monitoring of urine output is required, select a pre-connected, closed system urine meter tray.

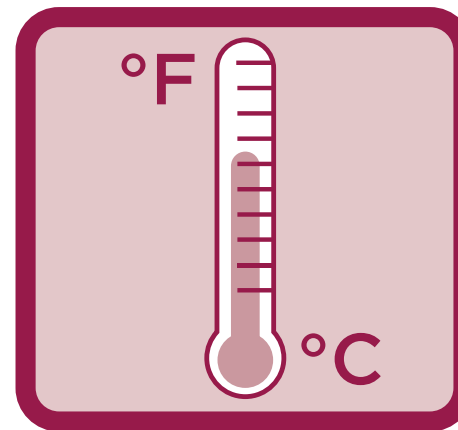

TEMPERATURE SENSING URINE METER TRAY

If the patient requires continuous temperature monitoring, a temperature sensing Foley catheter system may be selected.



NOT MADE WITH NATURAL RUBBER LATEX

If the patient has a latex allergy, choose a kit that displays this image.



No French Size?

If the label does not indicate the French size, the tray does **not** include a Foley catheter.

Foley catheters are intended for use in the drainage and/or collection and/or measurement of urine. The most common risk is urinary tract infection. Please consult product inserts and labels for indications, contraindications, hazards, warnings, cautions and directions for use. Refer to your facility's indwelling urethral catheter protocol for hospital-specific policies or procedures.