

On your mark, get ready. Flow!



Pre-Insertion

Obtain assistance PRN to facilitate appropriate visualization/insertion technique

Perform hand hygiene, don clean gloves

Perform peri-care, remove gloves, then, re-perform hand hygiene



Insertion

Maintain strict aseptic technique throughout the actual indwelling urinary catheter insertion procedure, re-perform hand hygiene upon completion

Foley catheters are intended for use in the drainage and/or collection and/or measurement of urine. Please consult product inserts and labels for indications, contraindications, hazards, warnings, cautions and directions for use.